HAYDi: Switch

Be a Part of Life

HAYDi:Switch provides physically challenged users with opportunities to work, learn, play and interact with others on their computer.

HAYDi:Switch is a combination of hardware and software solution. It allows a switch user to control a mouse, keyboard actions and making daily speech. Users need only a small movement like pressing or touching a switch button.

HAYDi:Switch has a second switch input port for special needs educators. Teachers, parents and carers can direct and work co-actively in directing, assisting or providing extra help for students, who use single switch education software.

Internet









Chat





E-Book

SHRYDI: SLUITER WIRELESS



Drawing



Cerebral Palsy

Multiple Sclerosis

Stroke

Writing



Game

Movie

Quadriplegia

Muscular Dystrophy

Call us now, let's find the best solution together ...

MS Bilgisayar Ltd.

Yargic Turhan Demirel Sk. No:7/8 34710 Kiziltoprak Istanbul TURKEY Tel/Fax: 0216 4184586

www.ms.com.tr





Windows" 1 2 MS Elipiaryar UM. www.ms.con.ti

HAYDI:SWITCH USB WIRELESS

Computer Interface **USB** Wireless Wireless Frequency & Type 2.4Ghz RF Operating Range 8+ Meter Number of Switch Inputs Switch Input Type 3.5mm mono male **Battery Operated** 6V - 4 x AA *Automatic Stand-By* 3 Minute Automatic Wake-Up 1. switch single click Stand-By Current & Time 1mA - 110 days **Operating Current & Time** 15mA - 7,5 days Average Battery Time 2 Months Transmitter Size 181x65x27 mm Transmitter Weight 110 Gr. (Without Batt.) Receiver Size 70x63x35 mm Receiver Weight 60 grams Receiver Cable Length 100 cm Receiver Operating Voltage 5V - USB powered

HAYDi:SWITCH USB

Computer Interface USB
No. of Switch Inputs 2
Switch Input Type 3.5mm mono male
Device Size 181x65x27 mm
Device Weight 135 grams
Cable Length 180 cm
Operating Voltage 5V - USB powered
Standards CE - RoHS - WEEE

System Requirements

Windows[™] Operating System XP - Vista - 7 1 Ghz or higher CPU 512KB or higher RAM 50MB free space on hard drive 1024x600 or higher resolution 1 available USB port Sound Card (for AAC) CD-ROM drive (for installation)

HAYDi:Switch Software

• With FQ/F+ and Mouse onscreen keyboards, you are able to use a PC with all features and applications

CE - RoHS - WEEE

• Other onscreen keyboards like Media Player, Internet Explorer and else, are designed especially to facilitate the usage of relevant applications

Standards

 HAYDi:Switch is able to create the link between applications and onscreen keyboards. For each application, it recalls automatically the linked onscreen keyboard
 You are free to designate the link between an application and an onscreen keyboard
 To facilitate and accelerate the

"target mouse" function on a large monitor (with e.g. 17inch or more screen width), you are able to divide the screen to 4 pieces and do the scanning in the area needed

You are able to perform sensitive mouse clicks by changing the settings of the smart mouse parameter
 HAYDi:Switch is flexible. It allows to define different usage features and settings for different users
 You can define as much user as your system allows. This feature is especially needed

in schools and multi user environments. Each user can work with it's own private settings • You can change the colours

of onscreen keyboards and indicator frames as you wish • On the Communication onscreen keyboard, besides 43 pre-recorded voice messages (male and female voices) there are 4 spare icons. These icons are especially created for your own recorded messages

Shift _



Yargic Turhan Demirel Sk. No:7/8 34710 Kiziltoprak Istanbul TURKEY Tel/Fax: 0216 4184586